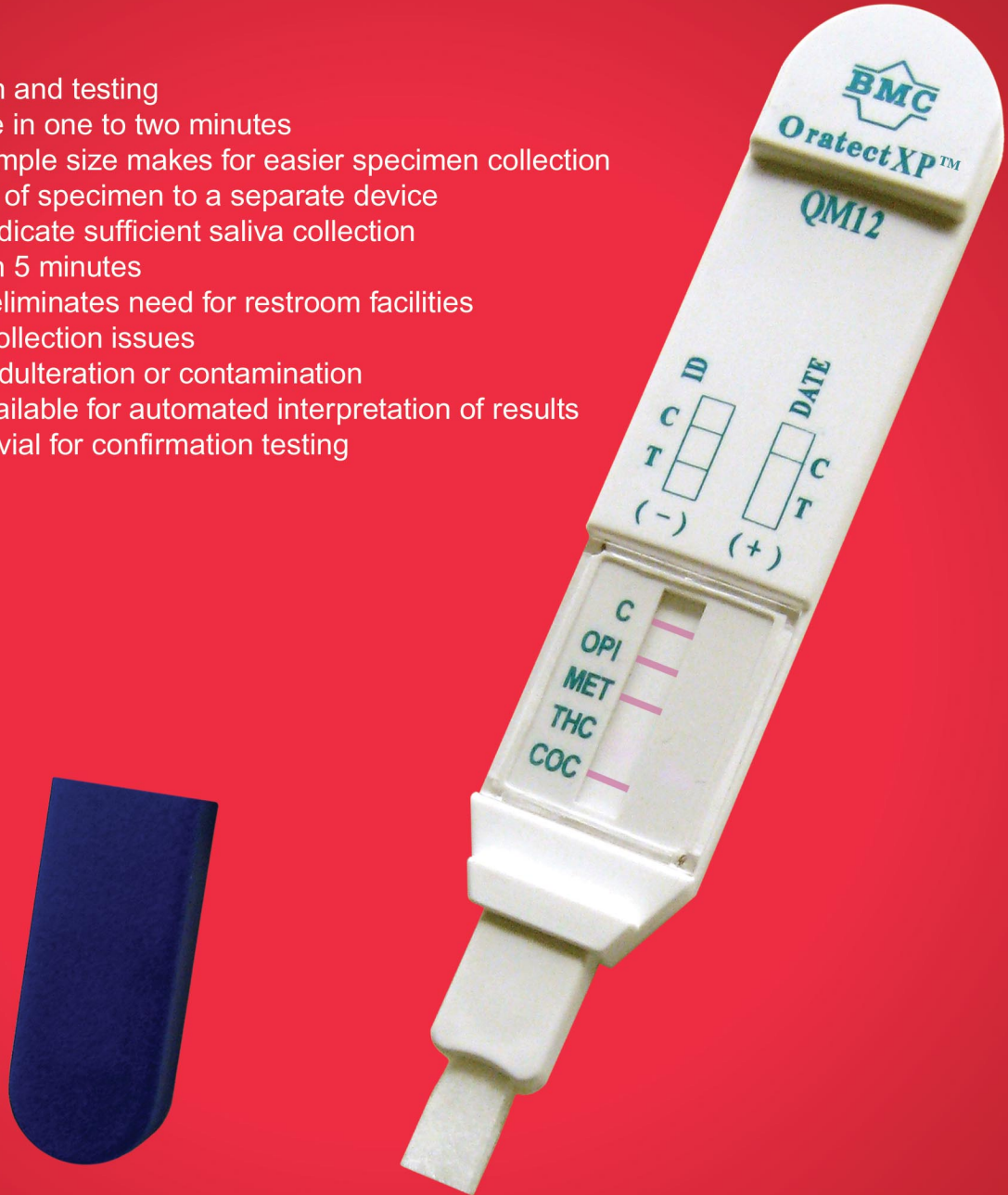


One-Step On-Site Device With Express Collection

OratectXP™

Oral Fluid Drug Screen Device

- One-step collection and testing
- Fast collection time in one to two minutes
- Reduced saliva sample size makes for easier specimen collection
- Eliminates transfer of specimen to a separate device
- Flow of blue line indicate sufficient saliva collection
- Results available in 5 minutes
- User-friendly and eliminates need for restroom facilities
- No cross-gender collection issues
- Reduces sample adulteration or contamination
- Portable reader available for automated interpretation of results
- Includes transport vial for confirmation testing



Drugs of Abuse Test Cut-off Levels:

Opiates (OPI)	10 ng/mL
Methamphetamines/Ecstasy (MET)	25 ng/mL
Marijuana* (THC)	40 ng/mL
Cocaine (COC)	10 ng/mL

*Parent THC



For Forensic Use Only







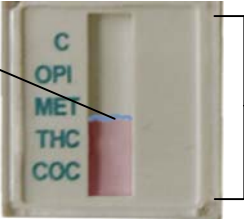
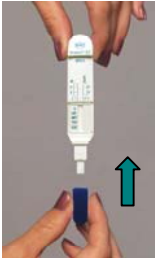
www.brananmedical.com






OratectXP™ Reference Guide

The OratectXP™ Oral Fluid Drug Screen Device is a simple one-step test for the detection of drugs of abuse in oral fluid.

OratectXP™ Procedures -- Refer to package insert for detailed instructions and technical information.

<p>1. Remove the blue cap by holding the sides and pulling gently. This will expose the collection pad. Make sure there is a blue line present in the window area indicating an unused device.</p> 	<p>2. Open mouth and gently rub the collection pad inside mouth against cheek in a circular motion several (approximately 15-20) times. Make sure to keep head level.</p> 	<p>3. Gently rub the collection pad against the opposite cheek in circular motion several (approximately 15-20) times.</p> 
<p>4. Gently rub the collection pad on top of the tongue several (approximately 15-20) times. Do not chew, suck, bite or bend the collection pad.</p> 	<p>6. Place the collection pad underneath the tongue for approximately 30 seconds to collect saliva. Instruct the donor to hold the device in place with hand. Sufficient amount of saliva collected is indicated by the flow of the blue line. Repeat steps 2-6 until the blue line flow.</p>  <p>Blue Line</p> <p>Test Window</p>	<p>7. Remove from mouth as soon as the blue line flow at the test window. Re-cap the device.</p>  <p>8. Lay the device on a flat surface and read results in 5 minutes after removing the device from mouth. Do not read results after 30 minutes.</p>

Interpreting OratectXP™ Test Results

<p>Invalid Result When no colored band appears in the CONTROL (C) region, the test is invalid even if there is a band in the test region. Repeat the test with a new device.</p> <p>Example Interpretation:</p>  <p>OPI: Invalid MET: Invalid THC: Invalid COC: Invalid</p>	<p>Negative Result Colored bands should be observed:</p> <ul style="list-style-type: none"> • One in the CONTROL (C) region • One in the specific TEST region <p>The color of the test band may be slightly darker or lighter than the control band. Any visible band that can be seen is a negative result.</p> <p>Example Interpretation:</p>  <p>OPI: Negative MET: Negative THC: Negative COC: Negative</p>	<p>Presumptive Positive Result A colored band at the CONTROL (C) region should be observed. When there is no colored band at the specific TEST region, the test is presumptive positive for that particular drug.</p> <p>Example Interpretation:</p>  <p>OPI: Negative MET: Presumptive Positive THC: Negative COC: Negative</p>
---	---	--