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**i-PAD NF1200 rescue**

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**Defibrillator**

The **i-PAD NF1200** is a semi-automated external defibrillator designed for minimally trained individuals. It provides simple and direct voice prompts and indications for a straightforward rescue operation. It is lightweight and battery powered for maximum portability.

The **i-PAD NF1200** is designed to treat Ventricular Fibrillation (VF) and Fast Ventricular Tachycardia. These two are the most common causes of sudden cardiac arrest (SCA). In SCA, the heart of the patient suddenly stops pumping. This condition occurs suddenly to any age group without any warning. The only effective treatment for VF is the application of a defibrillating shock.

**INDICATIONS FOR USE**

Use the **i-PAD NF1200** to treat a person that you think is suffering from sudden cardiac arrest (SCA).

The following are **symptoms of SCA**:

- No movement and no response when shaken
- No normal breathing
- No pulse

**REQUIRED USER TRAINING**

You must undergo training in the use of the **i-PAD NF1200**. You must also have training in cardiopulmonary resuscitation or other physician-authorized emergency response program.

**IMPORTANT**

The **i-PAD NF1200** may be used on children between 1 and 8 years old.

- If the patient appears to be from about 1 year of age to about 8 years of age, use the reduced-energy defibrillator pads.
- For children older than 8 years old, the American Heart Association (AHA) recommends using the adult Chain of Survival and resuscitation sequence (2005 AHA Guidelines for Cardiopulmonary Resuscitation and Emergency Cardiovascular Care).

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## i-PAD NF1200 rescue

**OPERATING MODE:** semi-automated

**WAVEFORM:** e-cube biphasic (truncated exponential type); impedance compensated

**ENERGY:** 200 J fixed as new norms

adult: nominal 150 J into a 50 Ω load

infant/child: nominal 50 joules into a 50 Ω load

**SHOCK ENERGY ACCURACY:** within ±3% (200 J into 50 Ω)

**SHOCK-TO-SHOCK CYCLE TIME:** typically less than 20 seconds

**PROTOCOL:** voice prompts and indicators guide user through protocol

**CHARGE CONTROL:** automatic by Software (arrhythmia detection system and charging control)

**CHARGE TIME FROM "SHOCK ADVISED":** <10 seconds, typical  
**VOICE INSTRUCTIONS:** detailed voice messages guide responder through use of the defibrillator

**CONTROLS:** shock Button, i-Button, On/Off Button

**INDICATORS:** 7 LEDs (different colors), i-Button

**CHARGE COMPLETE INDICATOR:**

- Text prompt (press the flashing orange button, now)

- Flashing backlight of SHOCK button

**DISARM:** Once charged, the i-PAD NF1200 disarms if:

- Patient's heart rhythm changes to non-shockable rhythm, or
- The SHOCK button is not pressed within 15 seconds after the i-PAD NF1200 is armed, or
- The Power ON/OFF button is pressed to turn OFF the i-PAD NF1200, or
- The defibrillator pads are removed from the patient or the pads connector is disconnected from the i-PAD NF1200

**SHOCK DELIVERY:** shock is delivered if the SHOCK button is pressed while the i-PAD NF1200 is armed

**SHOCK DELIVERY VECTOR:** via adult defibrillator pads in the anterior-anterior (Lead II) position or via reduced-energy pediatric pads in the anterior-posterior position

**PATIENT ISOLATION:** type BF

**BATTERY:** 12 Volt DC, 4.2 Ah, lithium manganese dioxide, disposable long-life primary cell (rechargeable battery with charger OPTION)

**BATTERY CAPACITY:** min.120 shocks or 4 hours of operating time

**AUTOMATED AND USER-ACTIVATED SELF-TESTS:**

- Daily Self-tests: tests internal circuitry, waveform delivery system, battery capacity, and software.
- Battery Insertion Test: upon battery insertion, extensive automatic self-tests and user-interactive test check device readiness.

**DATA RECORDING AND TRANSMISSION:**

- Infrared: wireless transmission of event data to personal computer using the IrDA protocol (SW+Irda cable option).

- Data stored: first 40 minutes of ECG and the entire incident's events and analysis decision.

**SIZE:** 2.75x8.66x10.23 (HxWxD)

**WEIGHT:** 2.2 kg with battery pack installed.